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First of all, a big Thank You for putting your trust in our product! It was developed with the sole aim of simplifying your everyday life with crutches or canes. True to our motto: *Getting well, not upset...!*

All our products are designed to be assembled in no time. If you still have difficulties, don't hesitate to ask your friends, neighbours or even complete strangers for help. They will certainly be happy to help you!

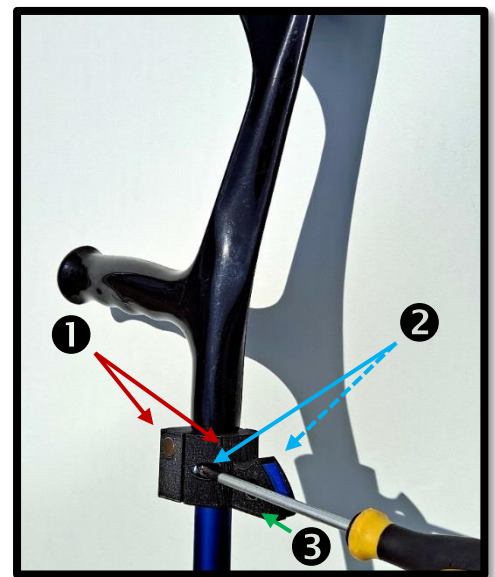
To avoid unnecessary waste and costs, we didn't include any tools. So please equip yourself with a medium crosshead screwdriver and you're ready to go:

Place the two halves of the **magnetic element** (❶) around the tube below the handle of the crutch or cane, hold them with one hand and tighten the two **screws** (❷) slightly.

Now turn the element in the desired direction, usually with the **"nose"** (❸) to the back.

Now tighten the screws all the way by hand until you feel resistance and the element no longer moves.

**ATTENTION:** Please do not tighten too much, it will hold...

### **Congratulations, you have made it!**

Now you have your crutches or your cane everywhere within reach, without constantly tripping over them... Hang them with the "nose" on the counter, on the edge of the table, on the chair or wherever else it suits you. And if you ever need to move a few metres with your hands free, just snap them onto the belt clip provided with the magnets and take them with you... Of course, you can also buy more belt clips and screw them on somewhere, e.g., on a wall, a wheelchair, etc. This way you can always keep things tidy.

### **Tip:**

The magnets also stick to many ferrous surfaces such as railings, fridges or even your neighbour's Rolls Royce...

### **And what about maintenance and warranty?**

Very simple: this product is completely maintenance-free. If necessary, you can wash your « Neo » with lukewarm water.

And we guarantee its quality for one year or 100'000 km (walked), whichever comes first...

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