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First of all, a big Thank You for putting your trust in our product! It was developed with the sole aim of simplifying your everyday life with crutches or canes. True to our motto: *Getting well, not upset...!*

All our products are designed to be assembled in no time. If you still have difficulties, don't hesitate to ask your friends, neighbours or even complete strangers for help. They will certainly be happy to help you!

To avoid unnecessary waste and costs, we didn't include any tools. So please equip yourself with a medium crosshead screwdriver and scissors and you're ready to go:

Control element:

IMPORTANT: Make sure that your crutches or your cane are adjusted to your height!



Place the two halves of the **control element** (❶) around the tube below the handle of the crutch or cane, hold them with one hand and tighten the two **screws** (❷) slightly.

Now turn the element so that the **"nose"** (❸) points backwards (for canes forwards) and the operating levers are parallel to the handle.

Move the rear **release lever** (❹) upwards until you see the **marking line** (❺) behind the **front retraction lever** (❻). It shows you the minimum length at which the release lever should not touch the handle (○). Move the whole element up or down until this is the case.

Now tighten the screws all the way by hand until you feel resistance and the element no longer moves.

ATTENTION: Please do not tighten too much, it will hold...

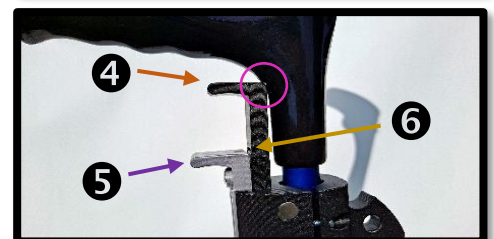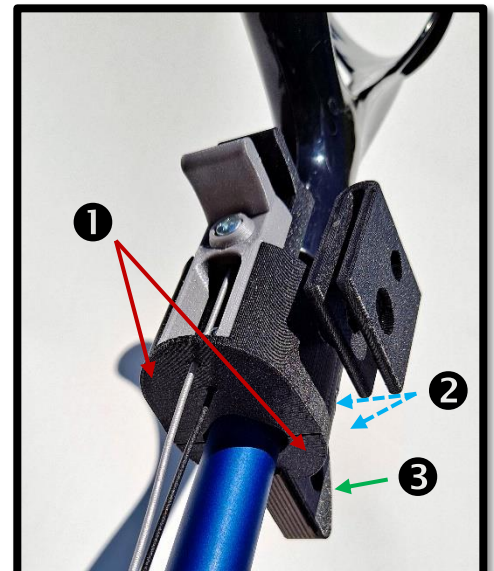
Stand element:

Place the two halves of the **stand element** (❶) around the tube above the rubber buffer, hold them with one hand and tighten the four **screws** (❷) lightly crosswise.

Now turn the element so that the **feet** (❸) point backwards. The coloured **retraction rubber** (❹) should now aim exactly at the "nose" (*).

The distance between the floor and the element should be around 6.5 cm, i.e. approximately its height. Now tighten the screws so that the element hardly moves. Manually extend the two feet and test the position of the crutch or cane. If it falls backwards, lower the element, if it falls forwards, raise it.

Now tighten the screws all the way by hand until you feel resistance and the element no longer moves.
ATTENTION: Please do not tighten too much, it will hold...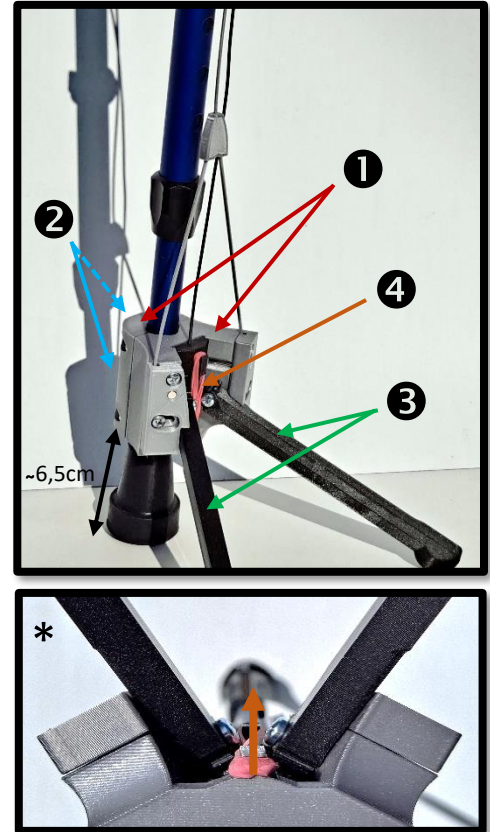


Release cable:

Make sure that the two feet are fully retracted.

Carefully pull the **release cable from the toothed glider** (❶) of the stand element half a turn around the stick and thread it from below into the **rear hole** (❷) of the operating element until it comes out again behind the **release lever** (❸). Make sure that the release lever is at the bottom.

Now tension the cable until the feet are about to be extended.

Tighten the **screw** (❹) in the release lever until you feel resistance and the cable can no longer be moved.
ATTENTION: Please do not tighten too much, it will hold. Please do not bend the cable.

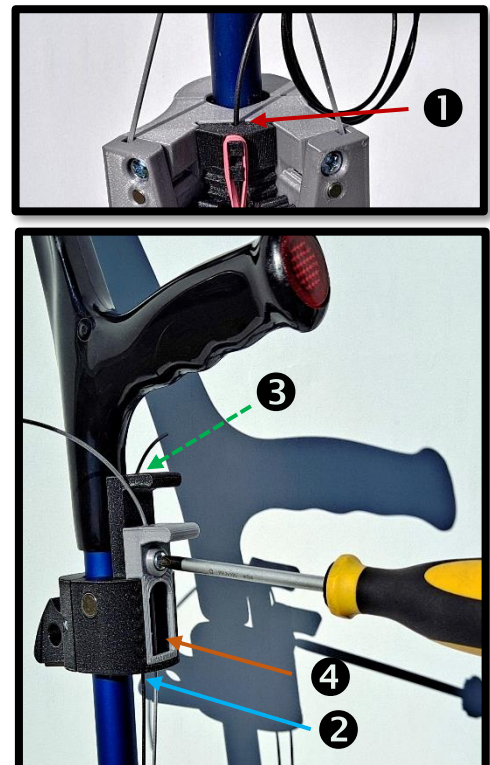
Retraction cable:

Make sure that the two feet are fully retracted.

Carefully pull the **retraction cable from the cable divider triangle** (❶) half a turn around the stick. At the same time, twist it once around the release cable (this will make them wrap nicely around the tube). Now thread it from below into the **front holes** (❷) of the control element until it comes out again above the **retraction lever** (❸). Make sure that the retraction lever is at the bottom.

Now tension the cable until the stand blockers are about to shear out into the sides.

Tighten the **screw** (❹) in the retraction lever until you feel resistance and the cable can no longer be moved. **ATTENTION:** Please do not tighten too much, it will hold. Please do not bend the cable.

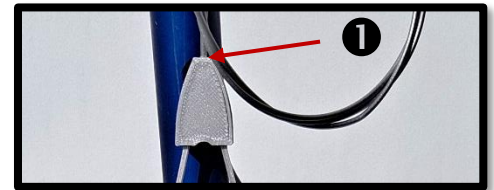Test:

Now test the engagement and disengagement of the feet, as well as the stance. If you are satisfied, you can now cut the rest of the cables behind or above the levers.

ATTENTION: Once you have shortened the cables, you can still shorten the length of your walking aid and adjust your « Neo+ », but you will have less room to lengthen them. Be sure that everything is the way you want it before you cut the cables.

In a pinch, you can always order new cables or install your own cable that is between 1.5 and 1.8mm in diameter (e.g. again 1.75mm 3D-printing filament or single-fibre fishing nylon). Heat the ends briefly with a lighter and compress, see original).

Congratulations, you have made it!

Now you have your crutches or your cane everywhere within reach, without constantly tripping over them... Put them down wherever you want, hang them with the "nose" on the counter, on the edge of the table, on the chair or wherever else it suits you. And if you ever need to move a few metres with your hands free, just snap them onto the belt clip provided with the magnets and take them with you... Of course, you can also buy more belt clips and screw them on somewhere, e.g., on a wall, a wheelchair, etc. This way you can always keep things tidy.

Tip:

The magnets also stick to many ferrous surfaces such as railings, fridges or even your neighbour's Rolls Royce...

If you want to fix the two elements even better to the tube, we have provided corresponding holes for this purpose.

And what about maintenance and warranty?

Actually, your « Neo+ » does not need any maintenance. However, if necessary, you can grease the moving parts (control levers, feet, foot blockers and toothed elements) with silicone oil, spray grease or similar. Caution: Do NOT grease the retraction rubber and remove it briefly beforehand, unless you use silicone oil. Do not grease screws, especially the threads in the plastic.

You can replace brittle or old retraction rubbers with the supplied replacement rubbers or normal office rubbers. Depending on the length and diameter, you may have to shorten them 1-2 times by folding them over. Test which of the two hooks on the toothed slider achieves the best tension. When installing the rubber, turn it around its own axis once so that the leg screws touch it less. If necessary, you can wash your « Neo+ » with lukewarm water.

And we guarantee its quality for one year or 100'000 km (walked), whichever comes first...

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